



Techno-Aide, Inc.
7117 Centennial Boulevard
Nashville, TN 37209
P: 800.251.2629
F: 800.554.6275

Weight Bearing Platform Stands CLEANING & CARE INSTRUCTIONS 2017

Thank you for choosing Techno-Aide for your radiology accessory needs. To ensure the longevity of your new Weight Bearing Platform, we recommend the following guidelines for cleaning and care.

Cleaning & Care:

We all know that in a medical environment, it is easy for surfaces to become dirty and soiled from blood, bodily fluids, betadine, barium, and etc.

Do:

- Techno-Aide recommends cleaning all surfaces with cold water and a mild detergent immediately after contact with any bodily fluids and after each patient use. Household 409 Spray can be used to clean the surface and remove dust and dirt.
- To disinfect the platform, stand and top, Techno-Aide recommends the use of disinfectant wipes such as PDI Sani-Cloth AF3 Alcohol Free Germicidal Wipes Cat# GWP-WAF. Allow a drying time of at least 3 minutes.
- PDI Sani-Cloth Bleach wipes can be used as a disinfectant if you strictly follow the manufactures guidelines. Failure to comply with these guidelines will result in void of warranty.
- Store your platform stand in a clean and dry area when not in use.

Don't:

- Do not use Bleach or any harsh chemicals on the weight bearing surface as it may discolor

Failure to adhere to the above recommendations will shorten the life of your product, promote discoloration of the surface if comfort type, and will possibly void your warranty. Techno-Aide provides a one-year warranty on workmanship and materials.

